

Instruction Manual

Aluminium Commode Chair



CMAL01MSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



1 Back Panel

2 Hand Grip

3 Seat Board

4 Bedpan

5 Foot Tube

6 Non-slip
Footrest Pad

Product Installation

Parts List:



Backrest Panel*1



Seat Board*1



Bedpan*1



Hand Grip*2



U-shaped Round Tube*2



Curved Round Tube*2



Foot Tube*4



Long Screws*8



Nut*4



Short Screws*4



Washer*12



Assembly Tools

Take out all products and accessories after opening the package.



Take out the seat board and backrest panel, align them with the connectors and press down firmly to clip into place. Take 2 short screws and tighten them at the corresponding positions, as shown in the diagram.




Take out the U-shaped round tube, align it with both sides of the seat. Place washers and long screws flat at the corresponding positions, then tighten to secure, as shown in the diagram.



Take out the four foot tubes, insert them into the four corresponding connectors respectively and clip them securely in place, as shown in the diagram.



Take out the two curved round tubes, place them between each pair of foot tubes respectively. Secure tightly with 2 washers, 1 nut and 1 long screw, as shown in the diagram.

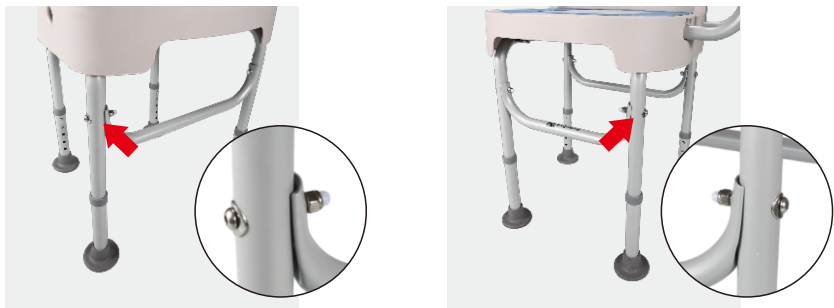

Take out the armrest, align it with the connectors on both sides, press and insert to install, as shown in the diagram.



Take out the commode pan, place it on the slide rails under the seat and push it inward, as shown in the diagram.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.