

Instruction Manual

Folding Aluminum Walker 3 in 1 function



CSW01MSIL

SectorCare

Contents

Safety Instructions	1
Product Overview	2
Product Installation	3-5

Safety Instructions

The walker **MUST** be level before use. Pinch points exist when installing and adjusted leg extensions. Ensure hands and body are clear of all pinch points.



CARE AND MAINTENANCE

Maintenance should be done at least every six months.
DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the product.



- 1 Regular cleaning is recommended. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
- 2 Regular inspection of parts including hardware, brackets, and plastics, for deformation, corrosion, breakage, wear or compression is recommended. Replace the product if any of these conditions exist.
- 3 If rubber tips are worn or cracked, replace them **IMMEDIATELY**.



Product Overview

Congratulations on your purchase of our products. Our walker are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.

Main Frame Parts



- | | | |
|----------------------------|--|--------------------------------|
| 1 Hand Grip | 2 Push-Button Folding Mechanism | 3 Commode seat |
| 4 Front Cross Brace | 5 Fixed/ Reciprocal Mechanism | 6 Height Adjustable Leg |

Product Installation

Setting the Walker

1. Remove the walker from the carton.
2. Swing both side frames outward from the front cross brace until an audible "click" is heard.



Folding the Walker

1. Locate the plastic locking piece on top of the front bar.
2. Press down on the locking mechanism and fold the side frames inward toward the front cross brace.



Adjusting Leg Extensions

1. Press the snap button, extend or retract the leg to the raise or lower the height.
2. Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole. (An audible "click" will be heard)
3. Make sure that the walker is level.



Commode Seat Assembly

1. Assemble the seat onto the frame.
2. Fasten the seat with bolts and nuts.
3. Lift the seat and lid, place the pail into the seat frame, then the walker can be used as a commode chair.



Fixed / Reciprocal Mode Adjustment

1. Press the folding button and fold the side frames slightly.
2. Fixed Mode: Press the snap button into the upper hole (an audible "click" will be heard).
3. Reciprocal Mode: Press the snap button into lower hole (An audible "click" will be heard).



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>