

Instruction Manual

Folding Aluminium Walker 3 in 1 Function



CSW02MSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



1 Hand Grip

2 Push-Button
Folding Mechanism

3 Commode Seat

4 Commode
Bucket

5 Fixed/ Reciprocal
Mechanism

6 Height Adjustable
Leg

Assembly Instructions



Main Frame x1



Toilet Seat x1



Toilet Lid x1



**Commode
Bucket x1**



**Hex
Screws x2**



**Hex
Nuts x2**



**Instruction
Manual x1**

Product Installation

1. Take out the product and accessories.




2. Unfold the Main Frame, lay the Toilet Seat flat on the frame, press down the clips to lock securely, as illustrated:



3. Insert Hex Screws and Hex Nuts into the clip positions on both sides of the Toilet Seat respectively and twist to tighten, as illustrated:



4. Lay the Toilet Lid flat on the seat plate, fit Commode Bucket to the bottom of the seat plate, as illustrated:

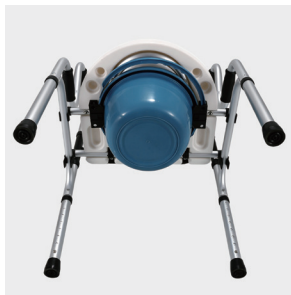

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.