

Instruction Manual

Folding Steel Bedrail



FBRST01WHI

SectorCare

<https://sectorcare.com.au>



Product Overview



1

Storage Bag

2

Foam Handle

3

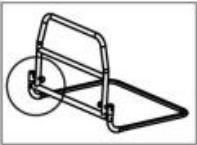
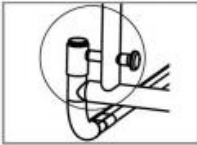
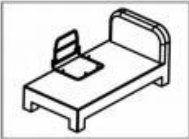
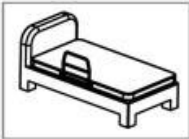
Folding Mechanism

4

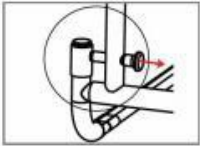
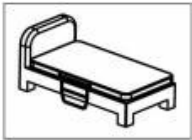
Steel Frame

Product Installation

Installation Procedure

1 Open the package, take the product out.	
2 Pull the bolt on both sides and unfold the bed rail.	
3 Both sides of the bolt must be inserted in the hole before use. Pass the seat belt through the bed and the bed rail, tie the rail to the bed, and adjust the length of the seat belt.	
4 Position the bed rail on the bed and secure it using a phillips screwdriver.	
5 Place the mattress over the base frame to complete installation.	

Folding Procedure

1 To fold the bed rail, pull the bolt on both sides.	
2 Rotate the bed rail outward simultaneously to fold it down.	
3 The bed rail is now in the folded position.	

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.