

Instruction Manual

Folding Aluminium Walker



FWALSSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



1 Hand Grip

2 Push-Button
Folding Mechanism

3 Main Frame

4 Front Cross
Brace

5 Fixed/Reciprocal
Mechanism

6 Height Adjustable
Leg

Product Installation

1. Take out the walker and all of the accessories from the carton.
2. Swing both side frames outward from the front cross brace and unfold frame completely until an audible "click" is heard.



Product Guidelines

Standard Walking Method

Lift the walker and take a step forward, then the user moves one step forward.



Reciprocal Walking Method

Lift up the left side and move forward one step, then user go one step forward.
Lift up the right side lifts forward and takes a step forward.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.