

Instruction Manual

Bath Chair with U-Shape Bench



SCAL01MSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



1 Back Panel

2 Seat Board

3 Foot Tube

Product Installation


Parts List:



Back Panel*1



Seat Board*1

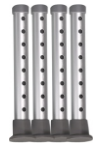

Foot Tube*4




U-shaped Round Tube*2




L-shaped Round Tube*2



Metal Washer*9

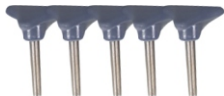

Short Triangular Head Screw*5

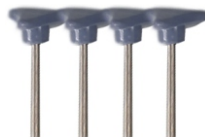

Long Triangular Head Screw*4


Take out all parts and accessories from the carton.



Flip the seat plate to the bottom and lay it flat on the ground. Take out two U-shaped round tubes, cross and place them flat in the grooves. Tighten with short triangular screws and gaskets at the intersection, as shown in the figure.



Take out four leg tubes, align each with the four connectors and insert until they clip securely into place, as shown in the figure.

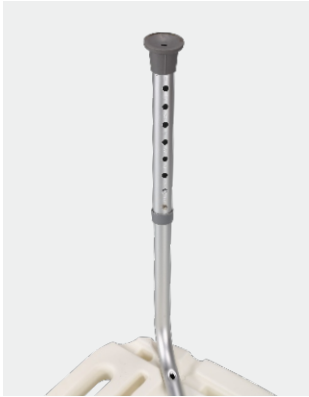

Take out the L-shaped round tubes, lay them flat on both sides of the intersection respectively, align with the connectors, then tighten and secure with long triangular screws and washers, as shown in the figure.



Take out the backrest plate, lay it flat on the two round tubes and align with the connectors, then tighten and secure with washers and short triangular screws, as shown in the figure.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.