

Instruction Manual

Bath Chair With U Shape Bench



SCAL01MSIL

SectorCare

Contents

Safety Instruction	1
Product Overview	2
Product Installation	3-4
Function Introduction	5

Safety Instruction

When using the product, basic safety precautions should always be followed:


- 1 Ensure all parts are properly assembled and securely tightened before use.
- 2 Use the transfer bench only on a flat, stable surface.
- 3 DO NOT exceed the maximum weight capacity.
- 4 Ensure all legs are adjusted to the same height and the bench is level before use.
- 5 Ensure rubber tips are intact and not worn or damaged. Replace if necessary.
- 6 Ensure the seat is securely locked before sitting or transferring.
- 7 Always sit down slowly and evenly to maintain balance.
- 8 DO NOT stand on the bench.
- 9 Use extra caution on wet or slippery surfaces.
- 10 Ensure the sliding/swivel seat is locked before attempting to stand up.
- 11 Keep hands clear of moving parts during sliding or swivelling to avoid injury.
- 12 Inspect the product regularly for wear or damage. Replace parts if necessary.
- 13 DO NOT use the bench on uneven surfaces or outside the bathtub setup.
- 14 DO NOT use on stairs or escalators.
- 15 Keep the product in a dry environment when not in use.



Product Overview

Shower chair Set Up Instructions

Please follow the step-by-step instructions on the following pages to properly assemble your sliding swivel transfer bench. The diagram below identifies the key components of the product to assist with assembly.

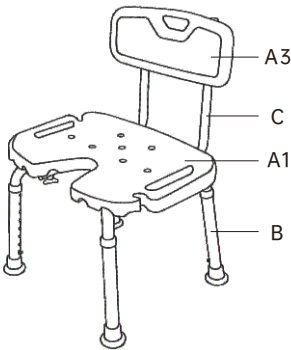

Part Number	Part Image	Description	Quantity
A1		Seat	1
A3		Backrest	1
B1		Leg Frame	1
B2		Leg Frame	1
C		Back Seat L-shape Frame	2
F1		Small Rubber Tip	4
G1		Short Screw	5
G2		Long Screw	4
G3		Washers	9

Product Installation


How To Install

Step 1:

Place Part A1 (seat) upside down on a flat surface so the screw holes are facing upward.

Align Part B1 (leg frame) with the seat, ensuring the concave side faces inward toward the center.


Align Part B2 (leg frame) on the opposite side, ensuring all screw holes are properly aligned.

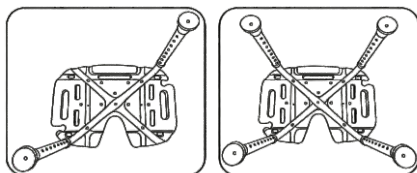

Step 2:

Align the center holes.


Insert the short screws (Part G1) with washers (Part G3) and tighten securely.

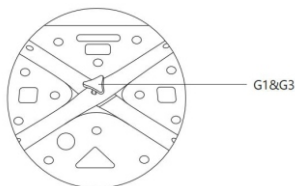

Step 3:


Place Part C (back support L-shaped frame) onto the leg frames, ensuring the screw holes are aligned. Insert the long screws (Part G2) with washers (Part G3) and tighten securely.

Repeat this step to install the second Part C on the opposite side.

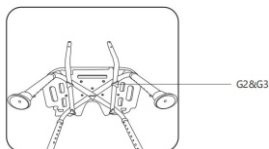

Step 4:

Attach Part A3 (backrest) to the back support frames (Part C), ensuring the screw holes are aligned. Insert the short screws (Part G1) with washers (Part G3) and tighten securely.

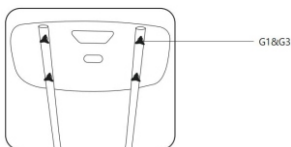

Step 5:

Check for stability. Ensure all screws are securely tightened and all legs are adjusted to the same height before use.



Function Introduction

Height Adjustment

The height can be adjusted to suit users of different heights. Press the push button on each leg while raising or lowering the leg tubes. Continue adjusting until the button clicks into the desired position.

Ensure all legs are set to the same height and are securely locked before use.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>