

Instruction Manual

Foldable Shower Bench with Wall Mount



SCAL02MSIL

SectorCare

Product Overview



- 1 Seat
- 2 Wall Mounting Bracket
- 3 Anti-slip Feet
- 4 Height Adjustable Legs

Assembly Instructions



Seat x1



Wall Mounting Bracket x2



Anti-slip Tips x2



Wall Mounting Screws x8

Product Installation

1. Remove the product from the packaging.



2. Press the seat board with one hand and pull the feet with the other hand to unfold the product.




3. Firstly, please adjust the installation position and the appropriate height before fixing the brackets.



4. Use a marker to mark the drilling position.



5. Drill holes (drilling bit diameter 8mm, hole depth not less than 5mm).

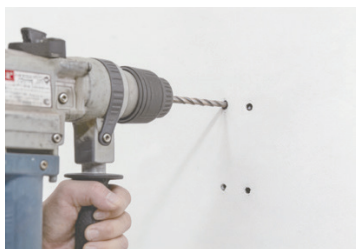

6. Insert all screws into the fixing plate.



7. Gently tap the screw into the holes with a hammer.



8. Use a wrench to tighten the screws.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>