

Instruction Manual

Bath Chair with Handle and Back



SCAL03SSIL

SectorCare

<https://sectorcare.com.au>



Product Overview



- 01 Backrest
- 02 U-shape Latches
- 03 Frame
- 04 Seat
- 05 Wheel

Assembly Instructions



Seat Board x1




Backrest x1



Main Frame x1



Front Castor x2

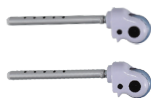


Rear Wheels
with Brakes x2



U-shaped
Clamps x4

Product Installation

1. Remove the product from the packaging.

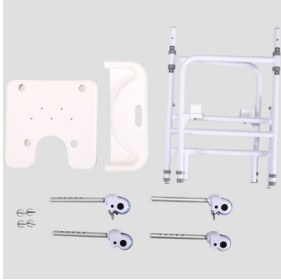

2. Unfold the main frame, then press down firmly on the latch to lock into place, as illustrated:




3. Position the backrest as shown, then press down firmly and insert to lock into place, as illustrated:



4. Place the seat board flat on the main frame, align with the grooves, then press down firmly to lock into place, as illustrated:



5. Take two rear wheels with brakes, align with the connection points on the rear leg tubes of the shower chair, insert and secure with U-shaped clamps. Then take two front castors, align with the connection points on the front leg tubes, insert and secure with U-shaped clamps, as illustrated.

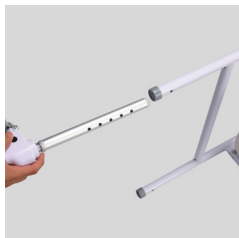

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.