

Instruction Manual

Aluminium Bath Stool



SSAL01MSIL

SectorCare

<https://sectorcare.com.au>




Product Overview



- 1 Seat
- 2 Height Adjustable Legs
- 3 Main Frame
- 4 Anti-slip Tips

Assembly Instructions

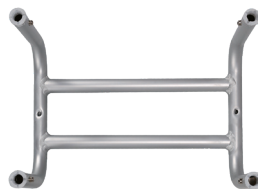

Main Frame Base x1



Seat Board x1



Leg Tubes x4



Triangle Screws x2




Curved Gasket x2



Instruction Manual x1

Product Installation

1. Remove the product from the packaging.

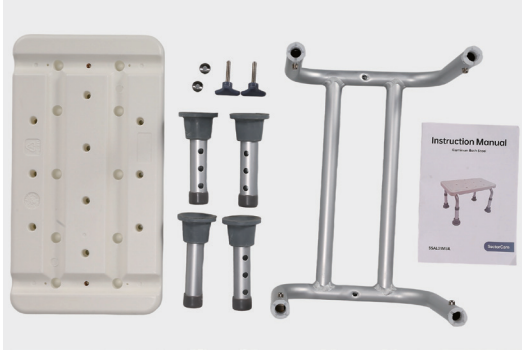

2. Align the Main Frame Base with the grooves on the back of Seat Board and lay it flat, insert Curved Gasket and Triangle Screws into the aligned joints and twist to tighten for fixation, as illustrated:



3. Insert Leg Tubes into the four connectors of Main Frame Base separately and secure them.



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.