

Instruction Manual

Toilet Aid with Suction Cup Caps




TSFST01WHI

SectorCare

<https://sectorcare.com.au>



Product Overview

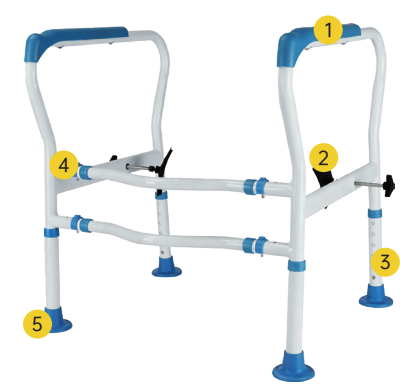

- 1 Handle
- 2 Toilet Clip
- 3 Adjustable Legs
- 4 Cross Support Frame
- 5 Anti-slip Foot Pads

Assembly Instructions



Main Frame x2



Curved Round Tubes x2



Triangle Screws x2



Fixing Washers x2



U-Clips x4



Instruction Manual x1

Product Installation

1. Remove the product from the packaging.



2. Align Curved Round Tubes with the ports of Arm Main Frame and insert them, secure with U-Clips, as illustrated:



3. Take out Triangle Screws, insert them into the connectors on both sides of Arm Main Frame, and secure with Black Plastic Rings at the other end, as illustrated:



LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.