

Instruction Manual

Manual Wheelchair



WCSP3

SectorCare

Contents

Introduction

- | | | |
|-----------|----------------------------|-------|
| 01 | Safety Instructions | 01 |
| 02 | Installation Guide | 02-03 |
| 03 | Usage Operation | 04-05 |

Safety Instructions

Please read this user manual carefully before using the manual wheelchair. Keep this manual for future reference. If you do not fully understand the instructions, warnings, or precautions in this manual, consult your dealer or a qualified technician before using the product. Improper use or incorrect installation may result in injury.



When using the product, basic safety precautions should always be followed:



- 1 Press seat cushion to lock crossbar into frame slots.
- 2 Fasten seat belt if physical control is limited.
- 3 Slow down on turns and slopes to prevent tipping.
- 4 Ensure both feet rest on the footrests when moving the wheelchair.
- 5 Keep fingers away from the top and bottom of the support tube to avoid injury.
- 6 When ascending stairs, the assistant should press the lower rear tilt lever and lower the push handle to lift the front wheels onto the stairs.
- 7 Use the brake to lock the wheelchair when parked to prevent movement.
- 8 Apply both brake handles together to slow down and maintain control.
- 9 Do not slide downhill or brake suddenly during fast driving.
- 10 Avoid stepping on pedals when entering or exiting to prevent tipping.
- 11 Do not overinflate tires to avoid discomfort or blowouts on bumps.



Installation Guide

Major Parts

Congratulations on your purchase of our products. Our Manual Wheelchair are manufactured from the finest materials and components available, and to meet our own highly rigid specifications and will surpass your expectations.



01 Back Push Handle

02 Brake Handle

03 PU Armrest

04 Breathable Seat Cushion

05 Parking Brake

06 Leg Protection Cushion

07 20 Inch Solid Back Wheel

08 Footrest



09 6 Inch Solid Front Wheel

Installation Procedure




1. Stand behind the wheelchair, firmly grasp the push handle, and lift the folding mechanism using the armrest for support. Skip this step if no folding mechanism is present.



2. Grasp each push handle and extend outward until the wheelchair is fully open.
3. Lower the seat cushion. If the wheelchair tilts, check seating and balance.

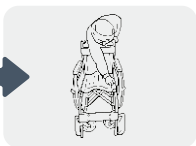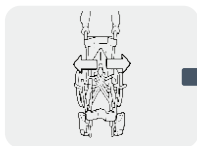




4. Set the footrest height, tighten bolts, rotate inward, adjust as needed, and lock in place.

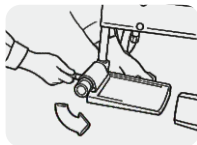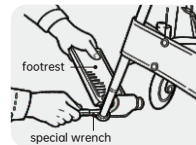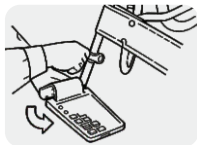

Usage Operation


Folding Procedure

1. Fold down both push handles, if applicable.
2. Fold up the foot pedal.
3. Lift the front center of the seat cushion to fold the wheelchair.

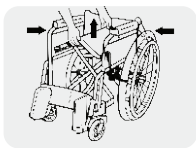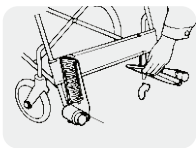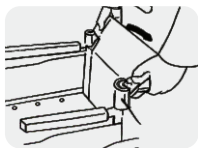

Method Of Getting On And Off

1. Lift the foot pedal, apply the brake, hold the armrest, and enter slowly. Set the pedal flat after sitting.
2. Park, apply brake, lift the pedal, hold the armrest, and exit slowly. Do not step on the pedal.

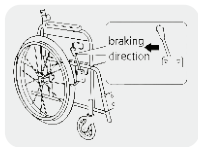

Shock Absorption System

1. The front shock absorption system absorbs bumps and vibrations, ensuring a smoother ride. It improves traction and braking on uneven terrain.
2. The rear shock absorption system enhances stability, reduces jolting, and lessens impact on the spine and back for greater comfort. It also improves performance on rough or uneven surfaces.

LEGAL STATEMENT

This manual and its contents are protected under the Copyright Act 1968 (Cth). No part of this publication may be reproduced, distributed, modified, stored in a retrieval system, or transmitted in any form or by any means without the prior written permission of FRG Pty Ltd. Unauthorized use is strictly prohibited.

© 2026 FRG Pty Ltd. All rights reserved.

SectorCare



<https://sectorcare.com.au>